

GeneXpert® System Maintenance Log for Instrument Serial Number 807

Canton City Public Health

FAS Installation Date: _____ Last Calibration Date: _____ Month/Year: _____/_____

Instructions:

1. Enter the current information in the fields above.
2. For each activity listed below check the box(es) under the day of the month that the activities were performed and initial in the bottom row.
3. Save the file after entering the data in the quality control log book.

Daily Manintenance	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
Clean Work Area	<input type="checkbox"/>																															
Close all module doors	<input type="checkbox"/>																															
Discard Used Cartridges	<input type="checkbox"/>																															
Weekly Manintenance	<input type="checkbox"/>																															
Power down the GeneXpert	<input type="checkbox"/>																															
Power down the computer	<input type="checkbox"/>																															
Monthly Maintenance	<input type="checkbox"/>																															
Archive tests	<input type="checkbox"/>																															
Purges test	<input type="checkbox"/>																															
Replace fan filters	<input type="checkbox"/>																															
Quarterly Maintenance	<input type="checkbox"/>																															
Clean Plunger rod & cartridge bays	<input type="checkbox"/>																															
Clean instrument surfaces	<input type="checkbox"/>																															
Test all GFCI receptacles	<input type="checkbox"/>																															
Yearly Maintenance	<input type="checkbox"/>																															
Annual instrument maintenance	<input type="checkbox"/>																															
As Necessary	<input type="checkbox"/>																															
Print System log report	<input type="checkbox"/>																															
Back up database	<input type="checkbox"/>																															
Technician Initials																																

1. Refer to Chapter 9 (Service and Maintenance) in the Operator Manual for detailed procedure.
2. Refer to Chapter 5 (Operating Instructions) in the Operator Manual for detailed procedure.

Reference: GeneXpert GX Dx Operator Manual (P/N 302-0528, Rev A) And Canton City Public Health Safety and Maintenance Manuals.